

**INSTALLATION AND ADJUSTMENT**

Read and follow all instructions, warnings, and notes in this manual before attempting to use your ramp for the first time. Your safety depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgment.



**WARNING!** Pride recommends installation on a flat, level surface such as cement. Avoid installing the ramp on gravel, sand, or any surface susceptible to erosion.

**Single-piece Ramp Installation Instructions**

Pride recommends that your Rubber Threshold Ramp be securely fastened to the surface upon which the Rubber Threshold Ramp is being installed prior to use. Use mounting screws or anchors (not supplied) to fasten the first section of the Rubber Threshold Ramp to the floor. Ensure that the ramp is free of dirt, debris, snow, and ice prior to use. See figure 1.

*NOTE: This product is recommended for outdoor use, but can also be used indoors.*

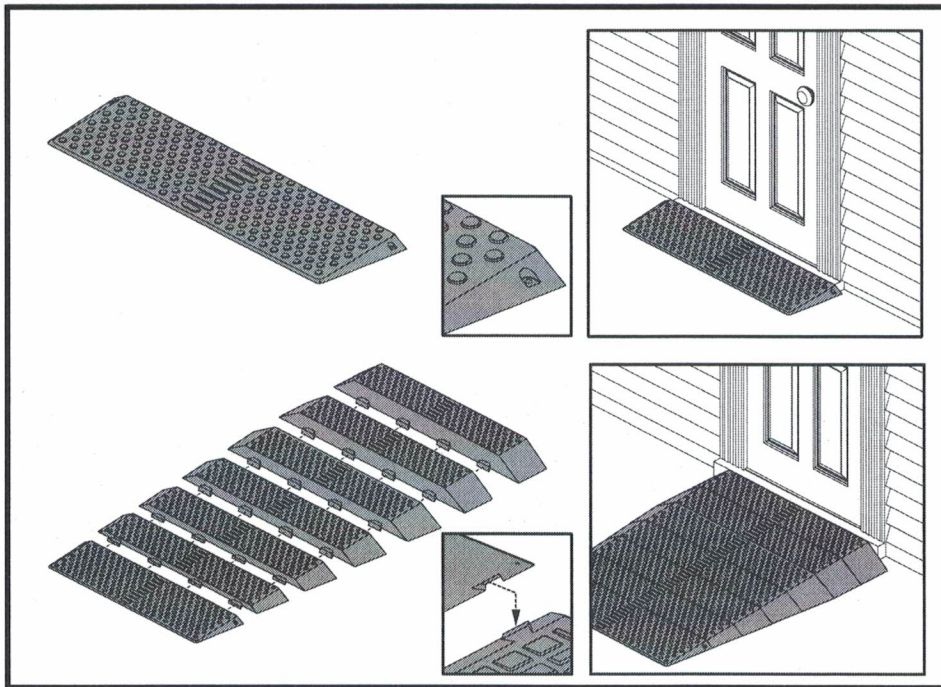


Figure 1. Rubber Ramp Threshold Installation